

Repair kit for travers lock (NT-820-20)


NODIN MEDEVAC STRETCHER (NT-820) NSN-6530-25-160 1429:









Version: 01
Issue: July 2019

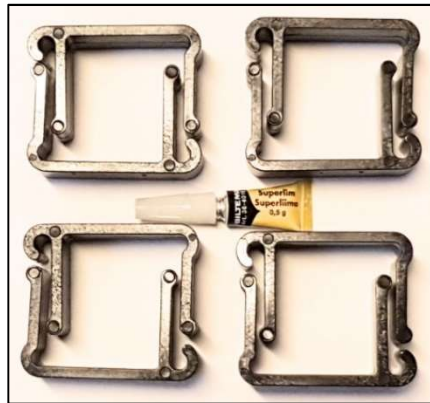

Single travers locking unit

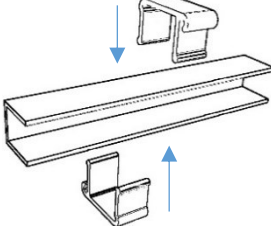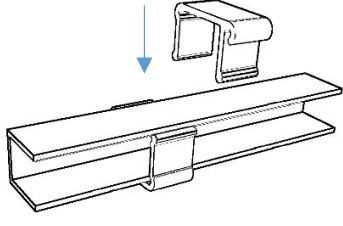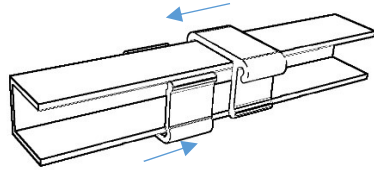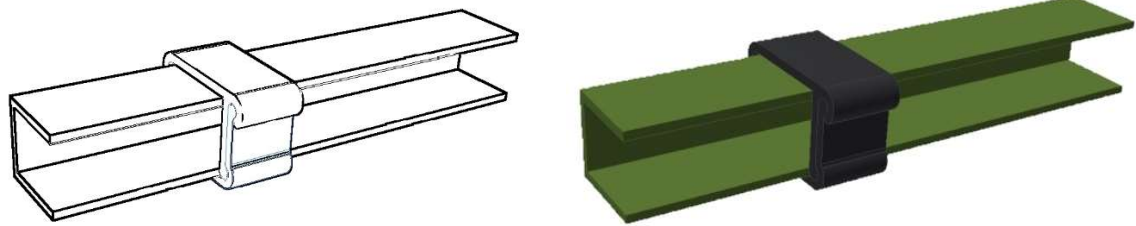

Complete locking kit for one stretcher

 <p>1: Place one part above the traverse and one part under the traverse, opposite to the other.</p>	 <p>2: Insert the parts on the traverse - one from underneath and one from the top.</p>	 <p>3: Position the parts so they can slide into each other.</p>
 <p>4: Press the parts together into one unit. The locking mechanism is fully assembled.</p>		
  <p>5: After the assembly, the two parts are fixed by applying the liquid adhesive along the two slots, as shown in the pictures above. Best results if the adhesive is allowed to cure for a day before being exposed to stress.</p>		

Post:
PO Box 573 Nanset
N-3252 Larvik
Norway

Office:
Faret 2
N-3271 Larvik
Norway

Tel: +(47) 93 04 87 79
Mob: +(47) 41 32 79 43
www.medevac.no

e-mail: post@medevac.no
Entrep.no.: 879 913 802
NATO-code (NCAGE) N4597