



Operators manual NT-822MAD – Stretcher mattress

NSN: 6530-25-161-8870

The mattress is designed for use with **NODIN MEDEVAC STRETCHER**, NSN-6530-25-160-2914

Before use

Open the buckles and unroll the mattress. Place the mattress on the stretcher with the pocket for pillow facing down in the head end. Pass the shoulder and groin safety belts through the holes in the mattress.



The mattress has two loops in the foot end to be attached around the telescopic handles on the stretcher.



Field stretcher, with mattress and 7-point safety belts, is then ready for use.

After use

Roll the mattress firmly from the head end and fasten with integrated fastening straps to the buckles on the back of the mattress.





Padding Pocket and Lifting Straps



At the back of the mattress, at the head end, there is a pocket where you can put a pillow or something soft to elevate the patient's head.

Product Label is attached to the inside of the pocket.



The mattress has four lifting straps on each side, for patient lifting.



The mattress is packed in a waterproof bag.
(If packed and closed properly, it can be used as floating element).



NODIN Aviation AS
Vangsveien 10
N-1814 ASKIM
NORWAY

post@medevac.no
www.medevac.no

