



NODIN AVIATION AS

YOUR NEEDS – OUR KNOWLEDGE – EQUALS INNOVATION

www.medevac.no

Operation manual, doc no: **OM-900SWC**

Product: **Wheeled Stretcher Carrier**

Part No: **NT-900SWC**

Wheeled Stretcher Carrier

US Patent US9,192,528 B1



CMC Medical Devices & Drugs S.L.
C/ Horacio Lengo, N18, CP29006,
Málaga-Spain
info@cmcmedicaldevices.com





GOLDEN SEASON

31 PLAYFAIR ROAD, GOLDEN SEASON BUILDING, SINGAPORE 387993

+65 68632228 WWW.GSEASON.COM



Assembly and Operation Manual

 <p>Wheeled Stretcher Carrier</p>	 <p>Insert wheel axle rod</p>	 <p>Locking lever secure rod in place</p>
 <p>Both wheels mounted</p>	 <p>Lift the handle side of the frame</p>	 <p>Pull out and turn both handles 90°</p>
 <p>Push crossbar until locked</p>	 <p>Use the handle to activate the carrier support legs</p>	 <p>Stretcher Carrier is ready for use</p>

How to secure the stretcher to the carrier:



Secure the stretcher to the carrier by moving clamp over the top of the stretcher poles and push down the locking lever until you can feel a stopping point or hear one click.

How to apply the brake:



To engage wheel lock, rotate the lock hand until fully engages against the wheel. (over-center locking)

When ready to move the patient:



Using foot, push down the end of the crossbar and make the foot prop rotate until it locks itself into place. Usually one person can pull the stretcher carrier with a patient by themselves on lever ground.

When the stretcher carrier is at rest, or when ready to remove the patient and stretcher:



Hold the stretcher handle with one hand, and rotate the carrier handle with the other, achieving the support legs in the correct position. Make sure the cross bar is right, and thereby locked